

QUICK START GUIDE FOR NETTI MOBILE



Enable joy of life

My-Netti.com

CONGRATULATIONS

on having become the owner of an advanced Netti Mobile powered wheelchair. If it is looked after properly, your wheelchair will reward you with many years of reliable service.

- Please take a few minutes to look through this quick start guide and the user manual. Both documents are there to help you get familiar with your Netti Mobile and to get the best out of it.

Your Netti Mobile is designed for indoor and outdoor driving. It's highly maneuverable with a small foot print, which makes it particularly suitable for driving indoors in confined spaces. It is also well suited to driving outdoors on paved roads or in public areas. You must however avoid off-road driving or climbing steep slopes with uneven surfaces.

GUIDELINES FOR USE

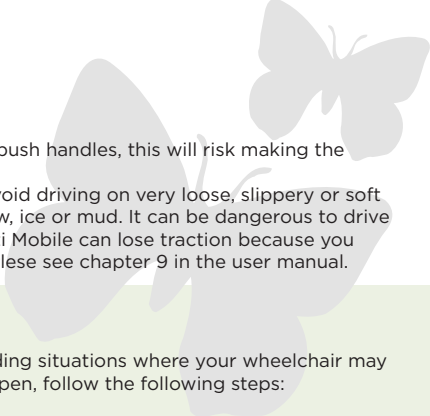
- **Charge the battery every night.**

Detailed instructions is found in chapter 10 in the User manual.

- Like all motorized electric vehicles the operating range of your Netti Mobile depends on how it used, where it is used and how well it is maintained. According to independent testing to international standards your Netti Mobile has a range of 25km on a full battery. You may, however find that the range in daily use is less than this. Factors such as ambient temperature, obstacles, and the type of surface will strongly influence the range you get.
- Always plan your journeys to avoid becoming stranded due to empty batteries. If ever the fuel gauge drops into the amber zone, we strongly recommend you consider returning home. Once the fuel gauge drops into the red zone there is a strong chance that if you continue to drive the chair, the batteries will be damaged irreversibly.

With that in mind you will reduce the chance of stranding and make your batteries last longer by taking the following precautions:

- Make sure your chair is fully charged before you start out on a journey.
- When crossing the road, look for dropped kerbs instead of attempting to climb them.
- **Make sure your tyres are properly inflated - 35 psi or 2,4 bar.**

- 
- Avoid carrying bags on the push handles, this will risk making the chair unstable.
 - Be very careful and try to avoid driving on very loose, slippery or soft surfaces such as gravel, snow, ice or mud. It can be dangerous to drive on surfaces where your Netti Mobile can lose traction because you may lose steering control. Please see chapter 9 in the user manual.

STUCK?

Although we recommend avoiding situations where your wheelchair may become stuck, if that does happen, follow the following steps:

- 1 Let go of the joystick.
- 2 Look around to find out what is preventing the wheelchair from moving. (There might be an obstacle blocking the front castors, unfavorable balance point of the wheelchair, slippery ground, water, wheelchair being attached to a docking station or strapped in place).
- 3 Try to improve the conditions so the drive wheels obtain as much traction as possible. Remove any loose obstacles and steer around any obstacles, walls etc. that cannot be moved.
- 4 Gently move the joystick in the direction that's most likely to free the wheelchair. If after 15 seconds, the wheelchair is not moving, the electrical system shuts down and the battery gauge blinks.
- 5 Let go of the joystick, wait for 30 seconds then try again, this time moving the joystick in a different direction.
- 6 If you are still trapped, go back to point 2, and have another look at the surroundings to sort out what is preventing the wheelchair from moving.
- 7 If you are still not able to move the wheelchair, get assistance.

WARNING!

The shut down of the system shows that the electrical system has been heavily loaded. Repeating this several times will lead to permanent motor damage.

Do not continue holding the joystick activated when the wheelchair cannot move. Failure to follow this advice can result in permanent damage to the motors. Damage caused in this way is not covered by warranty, nor does warranty cover driving under inappropriate conditions.

If Netti Mobile either becomes stuck or is used under arduous conditions outside of its specification, the motors and control system may become over heated. The control system will then switch off automatically.

If this occurs, allow the motors and control system to cool for at least 5 minutes before attempting to drive again. Any attempt to drive before the motors have cooled down, may result in overheating and permanent damage to them. Overheating damage to motors caused in this way is not covered by warranty.

THE LOCK

The Netti Mobile has an electronic lock to prevent unauthorized persons to start the wheelchair

TO LOCK

Press the on/off button 4 seconds when the chair is turned on. The chair is immediately turned off and the lock is activated.

TO UNLOCK

Press the on/off button.
Press the horn twice within 10 seconds. The chair is now unlocked.

In the full user manual there are more useful tips and instructions to help you understand and get the best use out of your Netti Mobile.

UM0117UK 2017-11

**WE WISH YOU MANY YEARS OF DRIVING
COMFORT FROM YOUR NETTI MOBILE**



Alu Rehab AS
Bedriftsvegen 23
N-4353 Klepp Stasjon
NORWAY

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry
DENMARK

Updates, user manual
and information are
found on our webpage:

My-Netti.com